

Measuring Instructions *CompreFit®/CompreFit® PLUS - LE (Lower Extremity)*

1. **BK** - Measure *length* from the **lateral** malleolus (**outer ankle bone**) to **lateral** aspect **Bottom of Patella**. Record in the corresponding **box G**.
2. Measure *circumferences* where indicated by the black dots; BK from the **lateral** malleolus (**Outer Ankle Bone**) - Record in the corresponding **lines A, B, C, C1**; TC from the **lateral** aspect **Top of Patella** - Record in the corresponding **lines D, E, F. (Right/Left)**.
3. Measure foot circumference and length and record on **lines I, J, K**. (Sizing Chart p. 36)

CompreFit® - TC Sizing Chart						
	Measuring Point	Small	Medium	Large	X-Large	XX-Large
F	23cm	48 - 58	56 - 66	64 - 74	74 - 84	n/a
E	15cm	43 - 53	51 - 61	58 - 68	69 - 79	
D	5cm	38 - 48	46 - 56	53 - 63	64 - 74	
∅ Point - Lateral Top of Patella						

CompreFit® - BK Sizing Chart						
	Measuring Point	Small	Medium	Large	X-Large	XX-Large
C1	30cm	29 - 39	34 - 44	39 - 49	48 - 58	55 - 65
C	25cm	29 - 39	34 - 44	39 - 49	48 - 58	55 - 65
B	15cm	24 - 34	29 - 39	33 - 43	41 - 51	44 - 55
A	5cm	20 - 29	21 - 30	25 - 36	32 - 42	33 - 43
G	Note: If BK length (G) is > 36cm order a size				TALL	
∅ Point - Lateral Malleolus						

CompreFit® PLUS - BK Sizing Chart						
	Measuring Point	Small	Medium	Large	X-Large	XX-Large
C1	30cm	24 - 34	29 - 39	34 - 44	43 - 53	50 - 60
C	25cm	24 - 34	29 - 39	34 - 44	43 - 53	50 - 60
B	15cm	19 - 29	24 - 34	28 - 38	36 - 46	39 - 50
A	5cm	15 - 24	16 - 25	20 - 31	27 - 37	28 - 38
G	Note: If BK length (G) is > 36cm order a size				TALL	
∅ Point - Lateral Malleolus						

CompreKnee™ Sizing Chart				
	Measuring Point	Small	Medium	Large-XL
D	5cm	38 - 48	46 - 56	53 - 74
C1	30cm	29 - 33	34 - 44	39 - 65
C	25cm	29 - 33	34 - 44	39 - 65