

Measuring Instructions *FoamSleeve™ - LE (Lower Extremity)*

- 1a. **BK** - Measure *length* from the **lateral** malleolus (**outer ankle bone**) to **lateral** aspect **Bottom of Patella**. Record in the corresponding **box G**. (*All measurements in cm*)
- 1b. **TH** - Measure *length* from the **lateral** malleolus (**outer ankle bone**) to **lateral** aspect **Gluteal Fold**. Record in the corresponding **box H**.
2. Measure *circumferences* where indicated by the black dots; BK & TH from the **lateral** malleolus (**Outer Ankle Bone**) - Record in the corresponding **lines A, B, C, C1**; TH from the **lateral** aspect **top of Patella** - Record in the corresponding **lines D, E, F**. (*Right/Left*).

FoamSleeve,™ Oversleeve™ - TH Sizing Chart								
	Measuring Point	Small	Small MAX	Medium	Medium MAX	Large	Large MAX	X-Large
F	23cm	48 - 58	56 - 66	56 - 66	64 - 74	64 - 74	74 - 84	74 - 84
E	15cm	43 - 53	51 - 61	51 - 61	58 - 68	58 - 68	69 - 79	69 - 79
D	5cm	38 - 48	46 - 56	46 - 56	53 - 63	53 - 63	64 - 74	64 - 74
∅ Point - Lateral Top of Patella								
C1	30cm	32 - 42		37 - 47		42 - 52		51 - 61
C	25cm	29 - 39		34 - 44		39 - 49		48 - 58
B	15cm	24 - 34		29 - 39		33 - 43		41 - 51
A	5cm	20 - 29		21 - 30		25 - 36		32 - 42
∅ Point - Lateral Malleolus								

TH - Length Sizes (cm) (Ankle to Gluteal Fold)				
	Extra Short	Short	Regular	Tall
H	51 - 61	61 - 71	71 - 81	81 - 91

FoamSleeve,™ Oversleeve™ - BK Sizing Chart						
	Measuring Point	Small	Medium	Large	X-Large	XX-Large
C1	30cm	32 - 42	37 - 47	42 - 52	51 - 61	58 - 68
C	25cm	29 - 39	34 - 44	39 - 49	48 - 58	55 - 65
B	15cm	24 - 34	29 - 39	33 - 43	41 - 51	44 - 55
A	5cm	20 - 29	21 - 30	25 - 36	32 - 42	33 - 43
∅ Point - Lateral Malleolus						

BK - Length Sizes (cm) (Outer Ankle to Bottom of Patella)		
	Regular	Tall
G	28 - 36	> 36