

Measuring Instructions

CompreBoot™ PLUS/CompreBoot™

1. Record *length* measurement **K** (length from 1st metatarsal head to heel or desired length) on the corresponding line. Select the boot size that matches according to **row K** on the Sizing Chart below.
2. Record *circumference* measurements **I, J,** and **L** on the corresponding lines.
3. Check to make certain that foot measurements **I, J,** and **L** *do not exceed* the adjustment range for the boot size selected in **STEP 1** (measurement **K** - Sizing Chart **row K**).
4. If any of the above foot measurements (**I, J,** and **L**) exceed the adjustment range of the boot size selected in **STEP 1 (K)** above, select the *next larger size* where all three measurements do not exceed the adjustment range of the boot.

CompreBoot™ PLUS/CompreBoot™ Foot Sizing Chart						
	Small	Small - Long	Med/Large	Med/Large - Long	XL-XXL	XL-XXL - Long
1. — K	14 - 18	19 - 23	18 - 20	23 - 25	20 - 22	25 - 27
2. {	I	28 - 36	28 - 36	39 max	39 max	44 max
	J	22 - 26	22 - 26	30 max	30 max	44 max
	L	25 - 30	25 - 30	36 max	36 max	43 max

Measuring Instructions

MedaBoot™

**Digital photos are required when irregular shapes are present*

1. Record *length* measurement **K** (length from 1st metatarsal head to heel or desired length) on the corresponding line.
2. Record *length* measurement **M** (Top of Foot/3rd Metatarsal Head to Ankle Bend) on the corresponding line.
3. Record *circumference* measurements **I, J,** and **L** on the corresponding lines.