

Measuring Instructions *Standard UE (Upper Extremity)*

1. Place the arm extended and relaxed slightly with the palm down on a flat surface.
2. Measure point A circumference at wrist. (All measurements in cm)
3. Measure point B circumference at elbow.
4. Measure point B1 circumference at mid bicep
5. Measure point C at axilla.
6. Measure D length at dorsal aspect from point A wrist to point C axilla.

CompreSleeve,™ MedaFit,™ FoamSleeve™ and OverSleeve™ Sizing Chart				
	A	B	B1	C
Small	14 -16.5	20 - 27	21.5 - 29.5	23 - 32
Medium	15 -18	24 - 30.5	26 - 34	28 - 37
Large	16.5 -19	28 - 35	30.5 - 39	33 - 43
XL	18 - 20	32 - 39	35 - 44	38 - 49

- (D) Order "Long" if length D \geq 48cm
- (D) Order "Regular" if 43 cm < length D < 48cm
- (D) Order "Short" if length D \leq 43cm

Hand/Glove Sizing Chart		
Size	Palm (E)	Wrist (A)
Small	15 -19	13.5 - 19.5
Medium	19 - 22	15.5 - 21.5
Large	22 - 26	17.5 - 23.5
XL	26 - 29	20 - 26